

## Technical Information

### Primax PT-F

### X-ray Film Tank Fixer (powder)

#### Description:

- Fixer for (manual) tank processing of all customary X - ray films
- powdery, one part

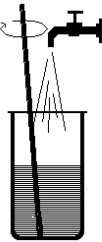
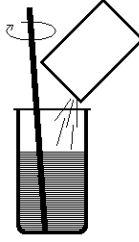
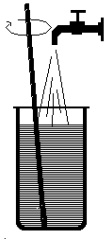
#### Processing Conditions:

fixing temperature :	18 °C	fixing time :	5 - 6 min.
	20 °C		4 - 5 min.
	22 °C		3 - 4 min.
	24 °C		2 - 3 min.

#### Replenishment:

For processing of 1 m<sup>2</sup> X - ray film **about 600 ml** tank solution are necessary at standard conditions .

#### Mix Instructions :

Water ( 30 - 40 °C )	PT-F - Powder	fill up with Water to
 about ¾ of tank		
7,5 l	Dissolve the powder under permanent stirring and then fill up to the final volume and mix well	10 l
17,0 l		22,5 l

#### Assortment :

Article Number	Product Name	Mix Size for	Capacity for ... m <sup>2</sup> film
209193	Primax PT-F	10 x 10 l	165
209197		5 x 22,5 l	175

#### Storage and Handling :

- Until the preparation PT-F has to be kept in the closed original package and stored at temperatures of 10 to 25 ° C
- All users of photographic processing solutions should exercise the greatest care to avoid the chemicals contacting the skin or eyes. Always wear solution resistant gloves.
- The disposal of used photographic processing solutions has to be done in accordance with the legal regulations.