

Technical Information

Primax LT- F

X-ray Film Tank Fixer (liquid)

Description :

- concentrate - fixer for (manual) tank processing of medical and technical X - ray films
- use in conjunction with the X - ray tank developer Primax LT- D preferably
- liquid concentrate, one part

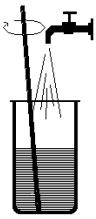
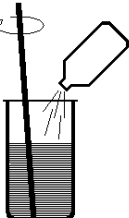
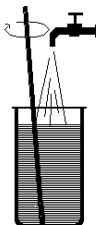
Processing Conditions :

fixing temperature :	18 °C	fixing time :	5 - 6 min.
	20 °C		4 - 5 min.
	22 °C		3 - 4 min.
	24 °C		2 - 3 min.

Replenishment :

For processing of 1 m² X - ray film **500 - 600 ml** tank solution are necessary at standard conditions.

Mix Instructions :

Water (25 – 30 °C)	Primax LT- F Concentrate	fill up with Water to
 about ¾ of tank volume		
19,0 l	Addition of the concentrate under permanent stirring	25,0 l tank solution or replenisher

Assortment :

Order Number	Product Name	Mix Size for	Capacity for ... m ² film
209207	Primax LT- F	2 x 25,0 l	80 - 100

Storage and Handling :

- Until the preparation Primax LT-F has to be kept in the closed original package and stored at temperatures of 10 to 25 ° C .
- All users of photographic processing solutions should exercise the greatest care to avoid the chemicals contacting the skin or eyes. Always wear solution resistant gloves.
- The disposal of used photographic processing solutions has to be done in accordance with the legal regulations.